

STERILIZATION QUICK GUIDE

ENSURE PROPER USE OF YOUR STERILIZER. FOLLOW THESE STEPS FOR STERILITY ASSURANCE, **EVERY TIME.**

STEAM STERILIZERS

Steam sterilizers are recommended as the sterilization method of choice by the CDC



GRAVITY DISPLACEMENT STEAM STERILIZER

Manufacturer Models Include: TUTTNAUER - 2340M, 2540M, 2540MK, 3870M, 1730 Valueklave

DYNAMIC AIR REMOVAL FOR STEAM FLUSH PRESSURE PULSE (SFPP)

Manufacturer Models Include: SCICAN - Statim 2000 / 5000 / 2000 G4 / 5000 G4, MIDMARK - M3 / M9 / M11, TUTTNAUER - EZ9 / EZ10 / EZ10K / 3870EA

1



Sure-Check® Pouches
For use with **EVERY LOAD**
(Internal/external multi-parameter chemical indicators)

2



STEAMPlus™ Class 5 Integrator
For use with **EVERY LOAD** or at least **EVERY DAY** for added sterility assurance

3



ConForm® Mail-In or In-Office
Spore test **AT LEAST WEEKLY**

DYNAMIC AIR REMOVAL PRE-VACUUM (CLASS B)


Pre-vacuum (Class B) – Requires daily air removal test (Bowie-Dick) to verify vacuum pump is working properly.
Manufacturer Models Include: SCICAN - Bravo, ADEC - Lisa, TUTTNAUER - Elara 11

1



Run Warm-up Cycle
with an empty chamber at the start of **EVERY DAY**

2

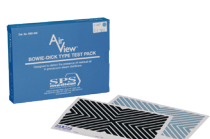

AirView™ Bowie-Dick Test
Use in empty chamber after warm-up cycle **EVERY DAY**

3



Sure-Check® Pouches
For use with **EVERY LOAD**
(Internal/external multi-parameter chemical indicators)

4




STEAMPlus™ Class 5 Integrator
For use with **EVERY LOAD** or at least **EVERY DAY** for added sterility assurance

5



ConForm® Mail-In or In-Office
Spore test **AT LEAST WEEKLY**

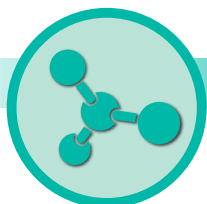

CHEMICAL VAPOR STERILIZERS

Chemical Vapor Sterilizers use formaldehyde solution instead of distilled water

Chemical-Vapor sterilizers look like, but do not act like steam sterilizers and are found primarily on the West Coast. Chemical-Vapor sterilizers are pre-set for 270F for 20 minutes exposure. No dry time is needed. Maximum load is 3.2lbs.

1



Sure-Check® Chemical Indicator Strips
For use in **EVERY POUCH**

2



In-Office SporView® Culture Set
Spore test **AT LEAST WEEKLY**

-OR-



ConForm® Mail-In
Spore test **AT LEAST WEEKLY**

DRY HEAT STERILIZERS

Used primarily in Ortho, Perio and Oral Surgery Practices

Dry Heat sterilizers are available in two different types – Convection (low cost and slow) or Rapid Heat Transfer (high cost and fast).

Convection: 320F for 1-2 hours depending on load size. **Rapid Heat Transfer:** 375F for 6 or 12 minutes, depending on the cycle selection.



1



Nylon Pouches with Class 1 Process Indicator Labels
For use with **EVERY LOAD.**

2



Dry Heat Chemical Indicator Strips
For use in **EVERY POUCH.**

3



In-Office SporView® Culture Set
Spore test **AT LEAST WEEKLY**

-OR-



ConForm® Mail-In
Spore test **AT LEAST WEEKLY**

Ordering Information
1-800-819-3336
customer@crossstex.com

Infection Control Helpline
TOLL FREE 1-855-878-3745

CROSSTEX
A CANTEL MEDICAL COMPANY

PROTECTS®

STERILIZATION QUICK GUIDE

Sure-Check® Sterilization Pouches

500/box

SCS25 | 3.5" x 9"

200/box

SCXX2 | 2.25" x 4"

SCX2 | 2.75" x 9"

SCXS2 | 3.5" x 5.25"

SCS2 | 3.5" x 9"

SCW2 | 5.25" x 6.5"

SCM2 | 5.25" x 10"

SCL5152 | 5.25" x 15"

SCL2 | 7.50" x 13"

SCL8162 | 8" x 16"

100/box

SCL10152 | 10" x 15"

SCL12152 | 12" x 15"

SCL12182 | 12" x 18"



Disposable AirView™ Bowie-Dick Test Pack

30/case
SBD-030



Sure-Check® Strips

100/bag
SCK | 3.75" x .625"



STEAMPlus™ Class 5 Integrators

100/bag

SSI-100 | 4" Strips



Dry Heat Nylon Self-Seal Pouches

100/box

NSP-400 | 2" x 10"

NSP-405 | 3" x 10"

NSP-410 | 4" x 10"

NSP-420 | 7" x 10.5"

NSP-430 | 9.5" x 13"

Class 1 Process Indicator Labels Included



ConForm® 10 In-Office Biological Monitoring System

C10SK

Digital Incubator and Record Keeper (Specify 110 or 120 Voltage)

C10B125

Steam Biological Indicators (25/box)



ConForm® 10

Dry Heat Chemical Indicator Strip

100/box

DIS-100 | 4" Strips



ConForm® Mail-In Biological Monitoring

Premium Test Service - Postage Included

CST060 | 6 Tests (3 Strip Test)

CST120 | 12 Tests (3 Strip Test)

CST480 | 48 Tests (3 Strip Test)

Value Test Service Tests

Postage Not Included

CVT120 | 12 (2 Strip Test)

CVT520 | 52 Tests (2 Strip Test)



SporView® Culture Set

SK-037 | SporView® Culture Set Starter Kit (Dry Heat & EO Gas)*

SK-056 | SporView® Culture Set Starter Kit (Chemical-Vapor)*

CS-020 | SporView® Culture Set (20 tubes and 20 spore strips)

CS-100 | SporView® Culture Set (100 tubes and 100 spore strips)



* Starter kits include box of 20 tubes/strips, dry block incubator (NDB601), record notebook and transfer rack to provide offices with all the necessary components for in-office testing.

Products manufactured for or by Crosstex - © 2014 Crosstex - 3/4/14 CTX1138

Ordering Information

1-800-819-3336

customer@crossstex.com

Infection Control Helpline

TOLL FREE 1-858-STERILE

CROSSTEX
A CANTEL MEDICAL COMPANY

PROTECTS®